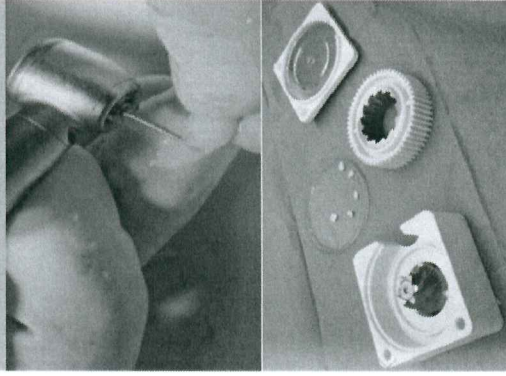
TT IN 10 STEPS

01

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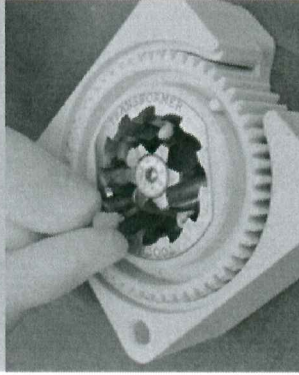
Prepare the tooth by drying it carefully using an air jet, remove any previous treatment (like root canals, filling etc.). You can use all teeth including the roots, nerves, you don't need to remove the enamel.

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
Insert the teeth into the Grinder MAX 3gr. Check the dimensions, optimal 5mm x 5mm. The grinder should be sterilized in autoclave. Average use of grinder is 100 interventions.

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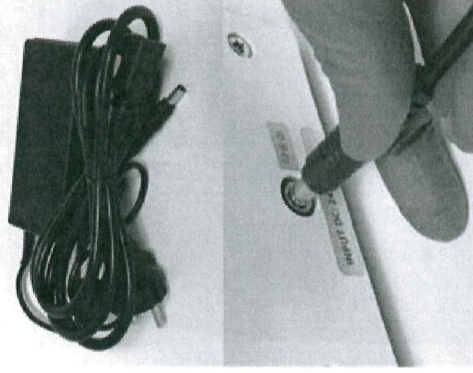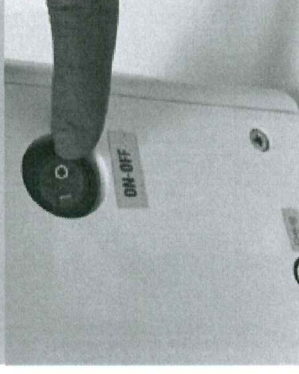
Switch on the TT by pressing the button on the rear of the device.

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
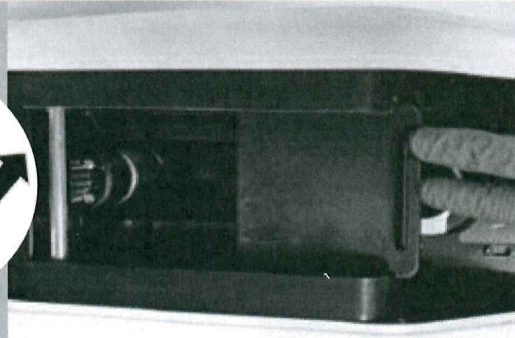
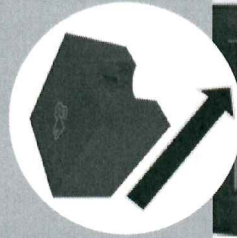
Insert the grinder, push gently until it reaches the end of the slide and gets locked into position, ONLY WHEN TT IS ON.

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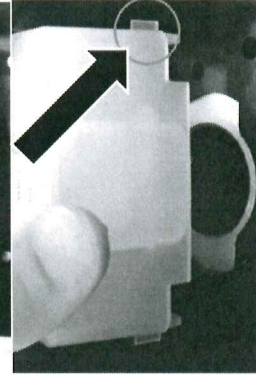
Insert the cartridge with liquids in the correct direction as shown in the picture (the sticker facing upward). Gently push the cartridge until it reaches the end of the slide.

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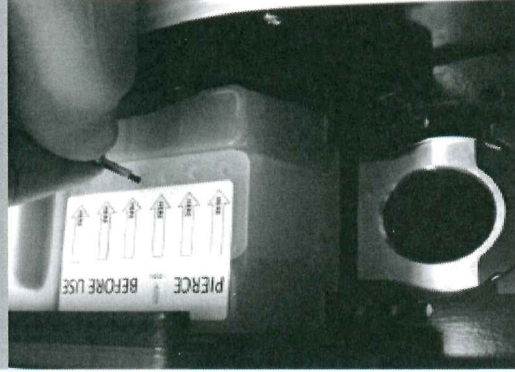Pierce the cartridge with a sharp sterilized instrument. The cartridge is for single use only, can not be used for more than one intervention. DO NOT pierce the cartridge at the bottom.

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Insert the CYLINDER with the MAKER into the slot. Make sure it is in the correct position and that it includes all its parts as shown in the image.

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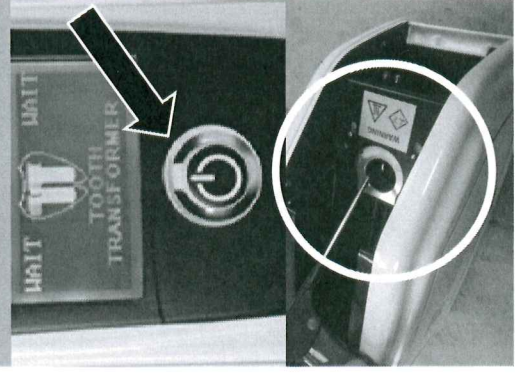
Close the lid and press firmly the START button to initiate the process.

Attention:
Do not forget to clean the TT with compressed air at the end of each cycle.

In case the TT does not work or errors appear on the display, we recommend repeating the procedure and watching the video tutorial on our website www.toothtransformer.com. After the first process phase, the TT will stop to give you the opportunity to check the content inside the grinder and the product present in the maker. After verification (all the teeth have been shredded and the product is present in the maker), you must start again from point 6. Duration of the cycle is between 20 to 25 min, this depends on the quantity of the teeth present in the grinder and their size. Preservation: it is possible to preserve the whole tooth, partial tooth or the granulate before the second phase, important that it is kept dry inside the TT Fairy. It guarantees patient safety. The material produced at the end of the cycle can be used within 3 hours. The regenerative material TT can be combined with other regenerative materials.